



DIAGNOSTIC SERVICES  
MANITOBA

SERVICES DIAGNOSTIC  
MANITOBA

<b>DSM Phlebotomy:</b> <b>Initials:</b> _____
<b>Accessioning Number:</b> _____ <div style="text-align: right;">_____</div> <div style="text-align: right;">Initials</div>

**PLEASE PRINT CLEARLY**

**PATIENT**

Last Name	First Name	Sex	Age	Date of Birth (dd/mm/yyyy)		

**ORDERING PHYSICIAN**

Last Name	First Name	Doctor Code

LifeLabs Genetics Counsyl Family Prep Screen	LOCATION CODE <b>LLGTCS</b>
--	--------------------------------

Collection Date: \_\_\_/\_\_\_/\_\_\_ Dy / Mo / Yr      Collection Time: \_\_\_\_\_

**Collection: by DSM Phlebotomist approved at the following sites:**

- Health Sciences Center
- St. Boniface Hospital
- Grace General Hospital
- Seven Oaks General Hospital
- Victoria General Hospital
- Westman Regional Lab

**REGISTRATION**

**REDR x 1**

**COLLECTION** (use in-house tubes)

1 – 4.0 mL Lavender top EDTA Tube

**PROCESSING**

**1 – 4.0 mL Lavender top EDTA Tube** - No Processing Required. **Ship AMBIENT on day of collection.**

**PACKAGING**

- Ensure tube is correctly labeled with patient’s full name and date of birth.
- Place blood specimen into the biohazard bag with absorbent and seal.
- Fold LifeLabs Genetics Counsyl Family Prep Screen requisition/ paperwork and place into pocket of the biohazard bag.
- Place the biohazard bag into the cardboard mailer.
- Close and seal cardboard mailer.
- Place mailer into FedEx shipment bag provided.
- Complete and attach the prepaid waybill.

**COURIER CONTACTS**

**FEDEX** – Samples **MUST BE** in the lab by 1345 to be shipped same day as collected.  
 For registration, processing and/or shipping questions, contact Laura Curtis, DSM Research Specialist 204-235-3935  
 For other inquires, contact LifeLabs Genetics at 1-844-363-4357